

SPINAL JACKET: INSTRUCTIONS AND CARE GUIDELINES

Single Patient Use Only-RX Only

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

Follow physicians and orthotist's instructions for length of wear.

CONTRAINDICATIONS:

Follow physician's and orthotist's instructions for contraindication and activity level while wearing the Spinal Jacket.

WARNINGS & PRECAUTIONS:

Physician's orders should supersede all protocol.

FITTING INSTRUCTIONS:



Step 1: Before donning make sure the patient is wearing a snug fitting T-shirt or stockinet. All wrinkles should be smoothed to prevent skin irritation

Step 2: Be sure to identify top and bottom before donning (look for National Labs label). While standing or lying down, align the Spinal jacket waist indentation to the patients waist groove.



Step 3: Palpate to find waist groove (just above pelvis and below lower ribs). This is the natural waist and anchoring point for the Spinal Jacket. Check with the physician to find out if the Spinal Jacket should be donned lying down or standing up



Step 4: When lying down slightly flex the hips and knees before tightening the straps. If standing slightly flex the hips and knee before tightening the straps.



Step 5: Tighten straps starting with the middle strap. Tighten straps as instructed or as needed to prevent the Spinal Jacket from moving on the patient.

CLEANING INSTRUCTIONS:

The Spinal Jacket may be cleaned with mild soap and warm water or wiped down with rubbing alcohol. Only use a towel for drying the Spinal Jacket. Do not use any harsh soaps, chemicals or detergents to clean the Spinal Jacket.

WARRANTY:

The device you received today is warranted to be made to your individual measurements, properly aligned and properly fit, corresponding to your anatomical condition at the time of measurement. Patient evaluation, consultation, designs, fitting and follow-up adjustments are provided for 90 days at no additional cost to you, unless there is a change in your physical condition. After 90 days, you are responsible for any charges for adjustments or modifications made to your device. Such services may be necessary for reasons such as changes in your physical condition, functional capabilities, or wear and tear or damage to your device. This warranty applies only to parts or devices that are custom made by National Labs Any part or device custom-made by National Labs is warranted to be free from defective workmanship and/or parts, under normal service and use, for 6 months from the date you receive it. National Labs' obligations under this warranty are limited solely to the no-charge repair or replacement of the part or parts determined defective by National Labs. Items and parts that are manufactured by outside companies or vendors are warranted for the length of the warranty supplied by that manufacturer. This warranty does not cover prosthetic skin coverings; adjustments needed due to anatomical or other medical changes; or accessories, such as prosthetic socks, straps, etc. Any claim whatsoever made by a recipient patient or reimbursing source/payer in connection with a device covered under this warranty shall be limited to the amount received by National Labs from the patient and/or reimbursing source/payer for the subject device. This warranty becomes void immediately, if: The device has been adjusted, repaired or altered by anyone other than an active employee of National Labs.

The device or any of its parts have been subjected to misuse, negligence or accident.

PATIENT RESPONSIBILITIES:

I understand it is my responsibility to:

See my prescribing physician if special medical management or further care pertaining to the wearing this device is necessary. Inform this facility should any adjustments to this device be needed. Keep all appointments, or contact this facility prior to the appointment time if rescheduling is necessary Inform this facility of any change in my overall health that may affect the wearing of this device. Inform this facility of all medications I am taking.

CAUTION:

Federal Law (U.S.A.) restricts this device to sale by or on the order of a licensed Health Care Professional.

Part Nos: F0381, F0391, F0411, F0421, F0550, F0560

© 2011 National Labs. National Labs is a trademark of SPS. All others are the properties of their respective owners.

National Labs-Tempe

1119 W Geneve Drive

Tempe, AZ 85282

Ph: 480.894.1755

Fx. 480.921.9686